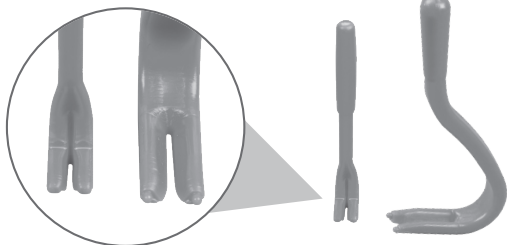
Zecken-Entferner-Set

Art.-Nr. 26104

DE	Gebrauchsanweisung	2
GB	Instruction manual	4
FR	Notice d'utilisation	6
ES	Manual de instrucciones	8
IT	Istruzioni per l'uso	10

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SET

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DE

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Art.-Nr. 26104

Stand: 01/17

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Status: 01/17

Dear customer,

thank you for deciding on one of our quality products. Please take your time to read through the manual and follow all safety and operational instructions in order to achieve best results. We would be delighted if you would recommend us and wish you success with the use of this device.

1. Product description

Note! Keep the Tick Removers and the packing materials away from children. The Tick Removers are small and can easily be swallowed by children!

The Gardigo Tick Remover will remove ticks easily and safely. In the packing there are two different sized hooks, allowing you to remove even small ticks and nymphs, without the danger of squeezing or irritating them. A tick bite can transfer a great number of diseases. The most common diseases are borreliosis, tick-borne encephalitis (TBE) and with dogs babesiosis. The germs are mostly in the salivary glands or the intestines of the tick and are inserted into the host's body through its hypostome whilst sucking blood. When removing a tick, it is essential that this is done swiftly, without irritating the insect or squashing it. This will reduce the danger of fluids from the tick entering your body.

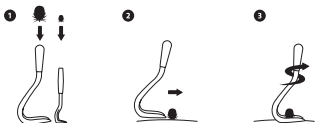
Please observe the following points:

- Remove a tick as soon as it is discovered. If any parts should remain in the skin, it is usually not the head but part of the jaws. Do not try to remove them, as this would open the wound can cause an infection. The leftovers will be rejected by the skin within a few days and do not pose a health hazard.
- Do not disinfect the Tick Remover before using it. Residue of the detergent can irritate tick.
- Do not try to remove the tick with a flame, liquids or oils.

- Do not squeeze the body of the tick.
- Do not remove a tick with your fingers or fingernails. The tick will inevitably be squeezed and there is the danger that the wound becomes infected.
- After you have removed the tick, store it in a closed container. Show it to your doctor as this will make a specification of the tick and the assessment of the danger of infection easier.

2. Application

It is important that the Tick Remover is placed as close to the skin as possible. This prevents that the tick is squeezed or irritated and that fluids and bacteria from the tick are released:

- Choose, depending on the size of the tick, one of the two Tick Removers.
- Lay the end of the Tick Remover flat to the skin, approach the tick from the side and try to wedge the head of the tick into the V-shaped tool.
- When fully inserted, turn the Tick Remover 1 – 2 times. By turning the tool, the barbed hooks of the tick's hypostome don't have so much grip and the tick is easily removed.
- Disinfect the wound and apply a plaster.
- Disinfect the Tick Remover and wash your hands thoroughly.
- Monitor the wound during the next days. If it becomes or stays red or begins to swell, please consult your doctor.

Dimensions: approx. 21 mm x 55 mm and approx. 8 mm x 47 mm

Weight: approx. 1,4 g and approx. 0,4 g

Material: Plastic

Please note, that illness or infection resulting from a tick bite or an incorrect or inappropriate use of the device, are not in the control of Gardigo. Therefore, Gardigo will assume no liability or responsibility for any health issues occurring. Please consult a doctor if you have been bitten or have any infection.

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